

# EXPORTING FROM **Sidexis XG**

---


UPLOAD GUIDE



---


866-497-3692 • [glidewelldental.com](http://glidewelldental.com)

Note: If using the Instrumentarium Dental OP300 or Gendex GXDP-700™, instructions will not apply. Please contact your local representative for assistance on extracting the raw data from your CT scanner.

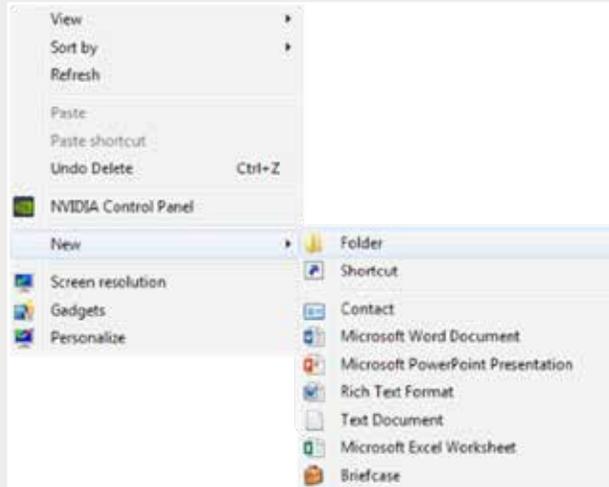


Step 1: Right-click on an empty space on your desktop to create a new folder ("New" → "Folder"). Type in the patient name and click "Enter".

1

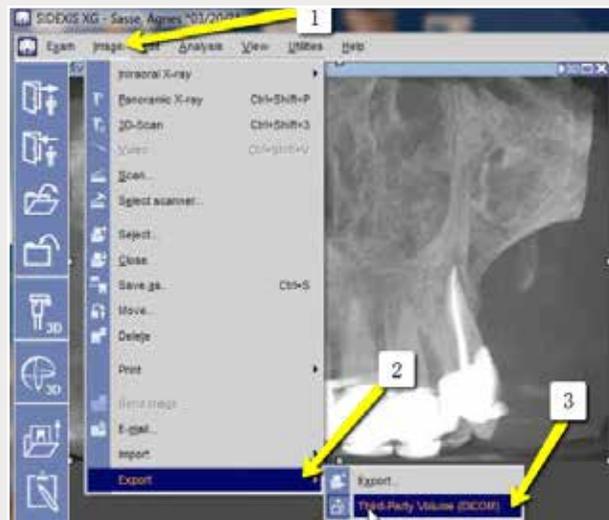


Step 2: Go to the patient's scan. Export from where the X-ray is located on the upper left corner. Select "Image" → "Export" → "Export Third Party DICOM".


Note: If using the Instrumentarium Dental OP300 or Gendex GXDP-700™, instructions will not apply. Please contact your local representative for assistance on extracting the raw data from your CT scanner.

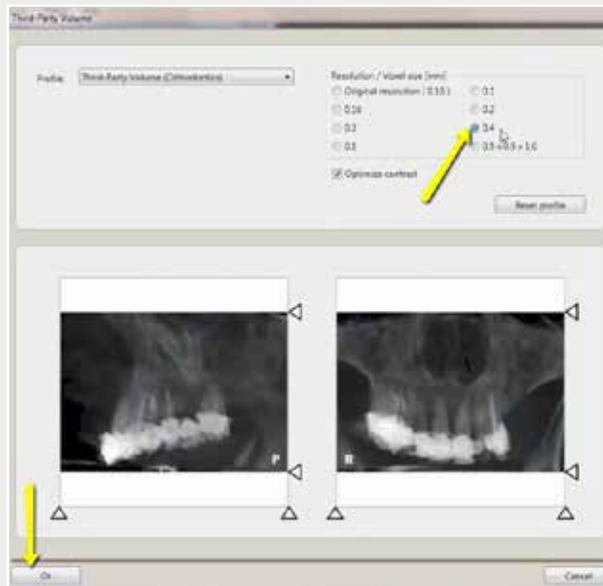


Step 3: Select "0.4" voxel size and click "OK".

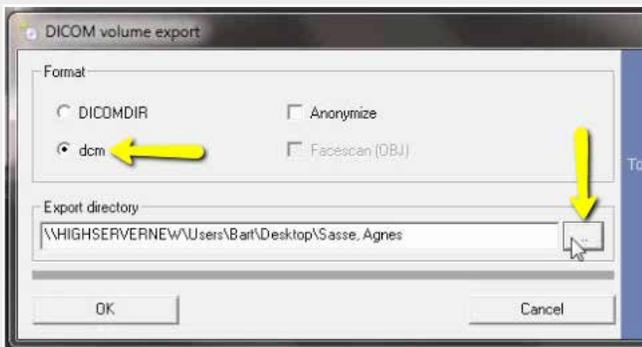


4a

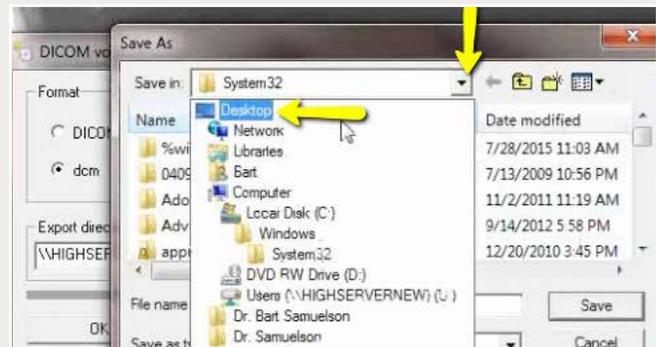

4b

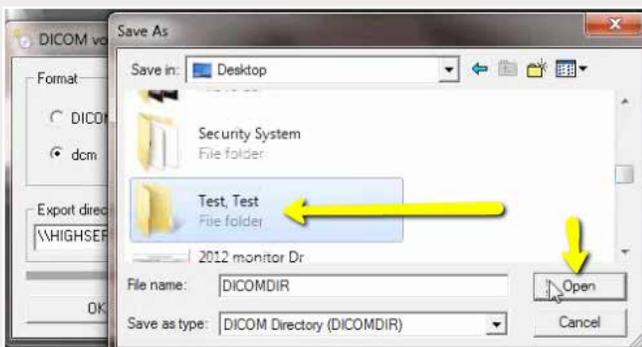

4c



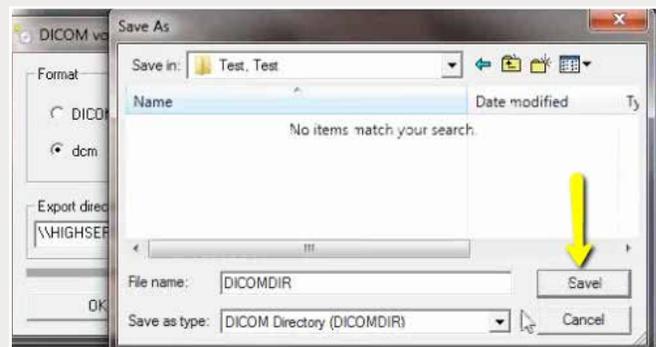


4d

Step 4: Select "dcm" under "Format" and click the "..." button to select the newly created folder of the patient on your desktop. Click "Open", then click "Save".


2

Note: If using the Instrumentarium Dental OP300 or Gendex GXDP-700™, instructions will not apply. Please contact your local representative for assistance on extracting the raw data from your CT scanner.

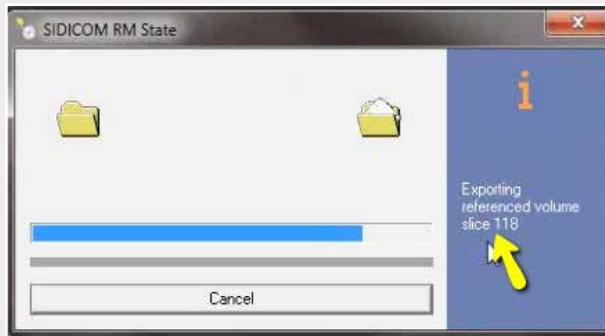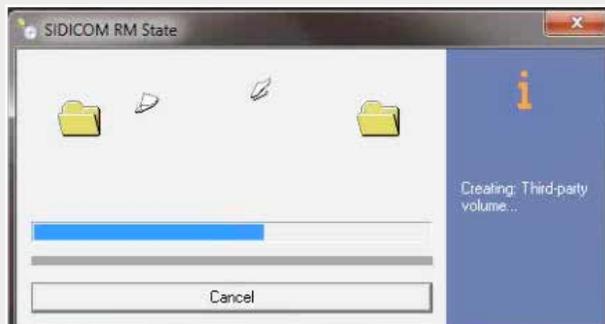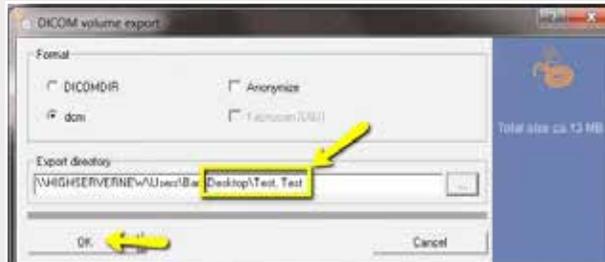

Step 5: Confirm the patient folder has been selected and click “OK”, and exporting will begin.


Note: If using the Instrumentarium Dental OP300 or Gendex GXDP-700™, instructions will not apply. Please contact your local representative for assistance on extracting the raw data from your CT scanner.

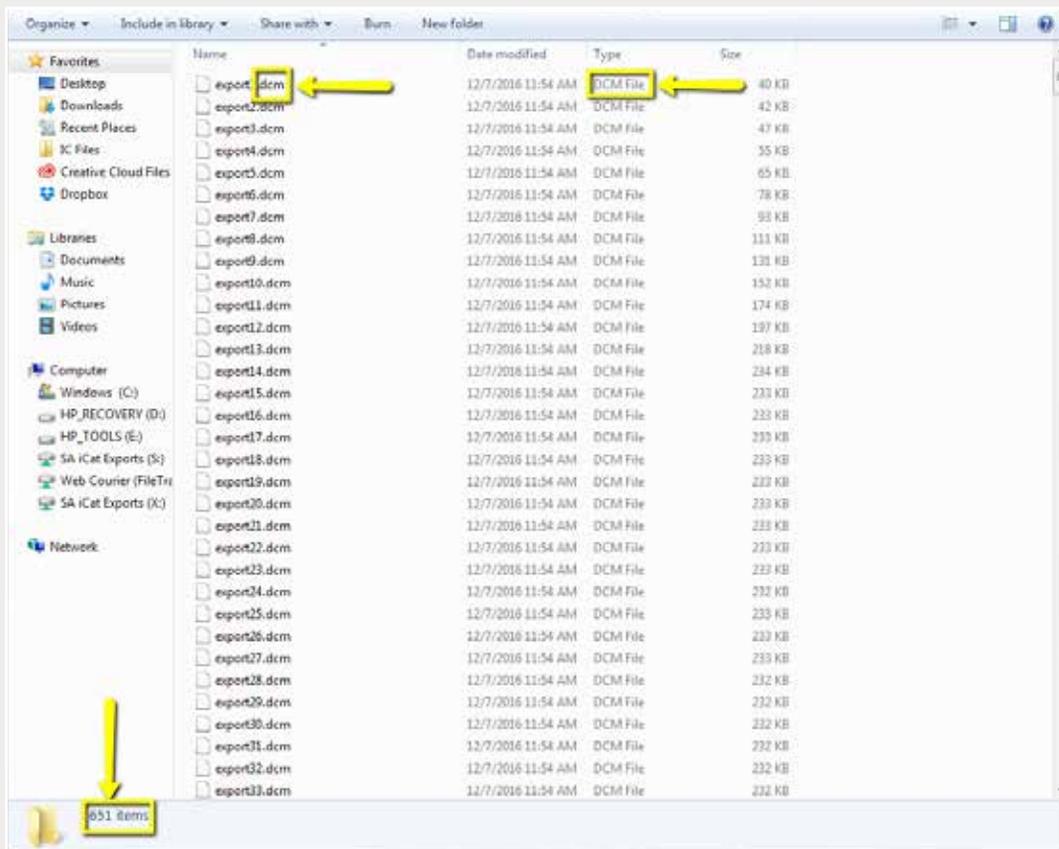

4

Step 6: A window will appear with DICOM data. Confirm the .dcm files were exported correctly (there should be several hundred files).

Note: If using the Instrumentarium Dental OP300 or Gendex GXDP-700™, instructions will not apply. Please contact your local representative for assistance on extracting the raw data from your CT scanner.

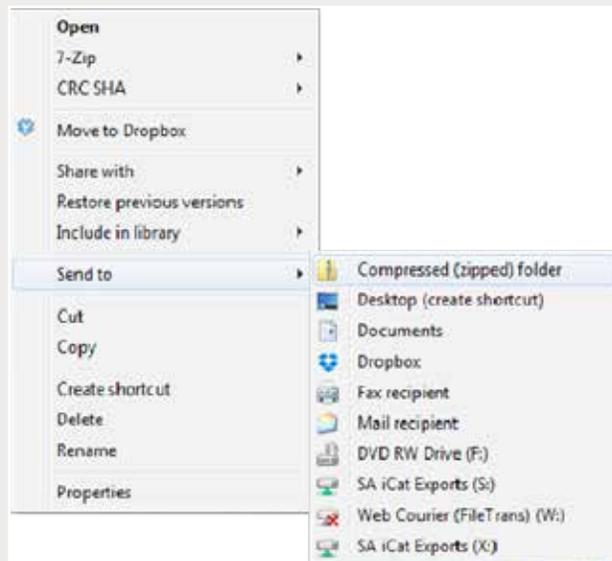

Step 7: Return to your desktop and right-click on the patient's folder to send to a compressed folder ("Send to" → "Compressed (zipped) folder"). A duplicate folder will be created with either a zipper or a blue "Z".

Note: If following a Dual Scan Protocol, zip scans individually and label accordingly (i.e., "Patient Scan" and "Denture Scan").

5



Step 8: Upload your case on [glidewelldental.com](http://glidewelldental.com), then click and log in to "My Account".

Note: Use Google Chrome when uploading your case; additionally, if you do not have an account, please contact us and we will set it up for you. Your scan can also be sent on a disc or USB drive via standard shipping methods if digital upload is not possible.


Note: If using the Instrumentarium Dental OP300 or Gendex GXDP-700™, instructions will not apply. Please contact your local representative for assistance on extracting the raw data from your CT scanner.



Step 9: Select "Cases" → "Guided Surgery/DTP".



Step 10: Select "Request New Plan".

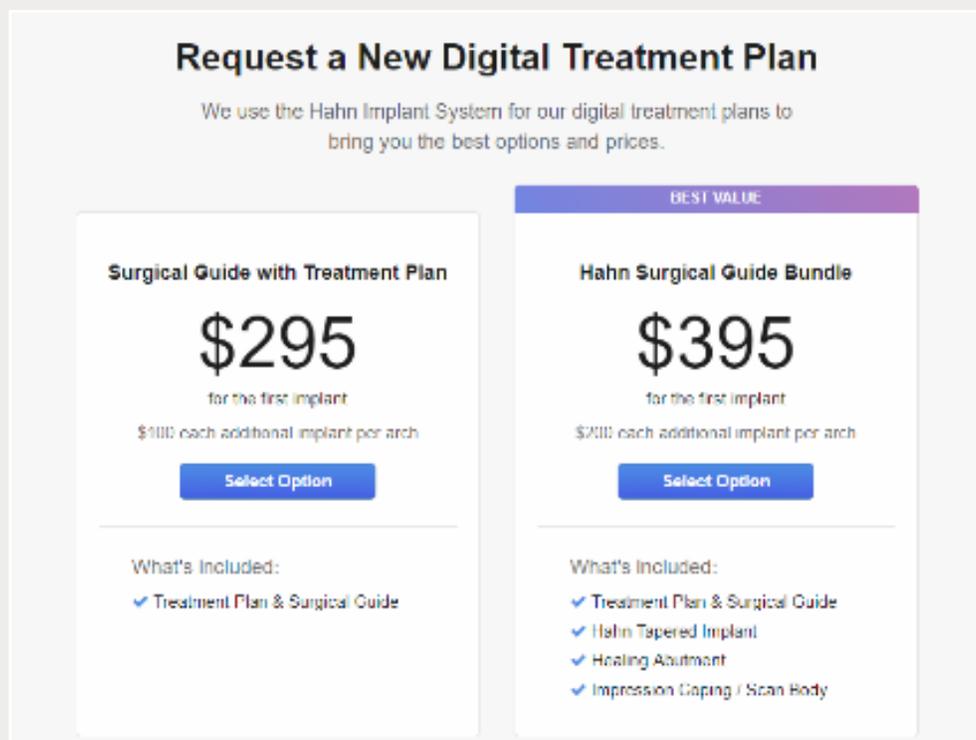

Step 11: Select desired package.

