


# ORBSPORT™

SMART MOUTHGUARD

QUICK START GUIDE

## WHAT'S INSIDE THE BOX? 1

1. ORB Sport™ Smart Mouthguard
2. Charging Case
3. Case Sleeve
4. USB to Micro-USB Cable

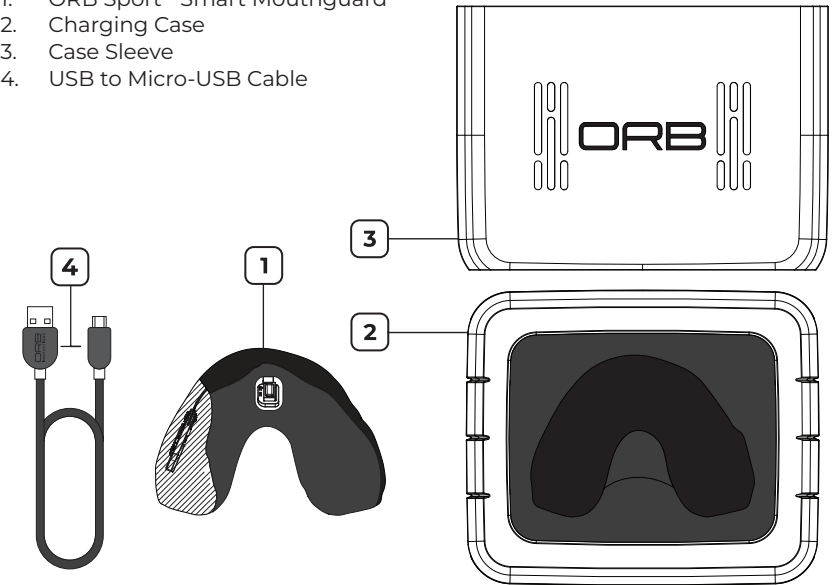


## INTRODUCING ORB SPORT MOUTHGUARD 2

Optical Heart Rate Sensor

Bluetooth® LE

Gyroscope


Wireless Charging Coil

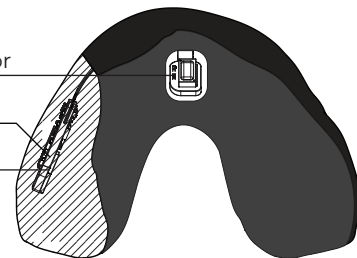

Indicator LED

Accelerometer

Memory Storage

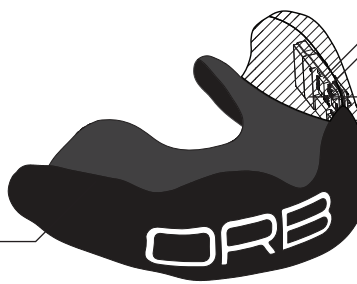



Bluetooth is a registered trademark of Bluetooth SIG, Inc.

## DOWNLOAD THE ORB SPORT APP 3

Scan the QR code and download the ORB Sport app to configure and manage your mouthguard. Access all of its enhanced functions today!


Use the ORB Sport app to set up:

- Sessions
- Trends
- Profile
- Device Settings
- ... and more!

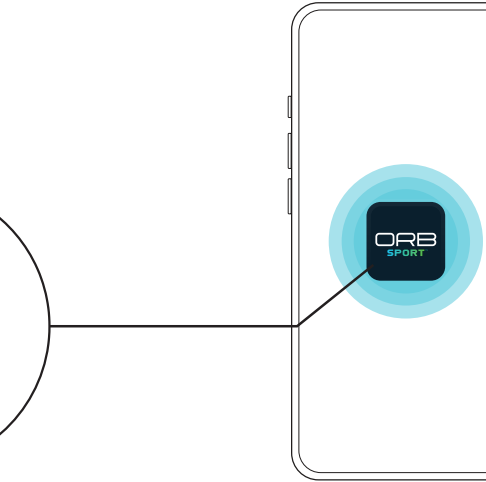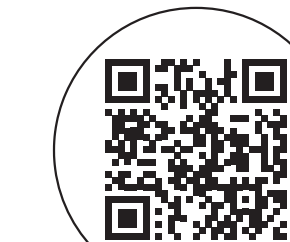

## CHARGING 4

Place the ORB Sport mouthguard into the Charging Case and connect to a USB power supply.

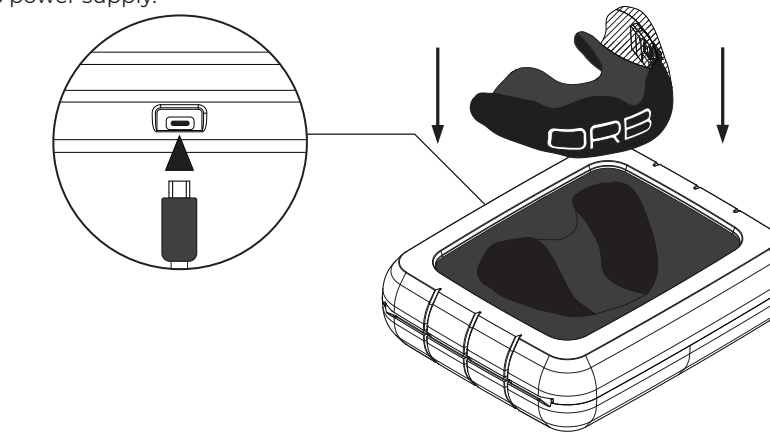

**!** Charge your ORB Sport mouthguard for at least 1 hour before use.


**NOTE:** Please ensure the Charging Case is connected to a USB power supply, otherwise it will not charge. If an ORB Sport mouthguard is being charged from a depleted battery, the red indicator LED will rapidly blink for 1 second to indicate the product has turned on.



## CHARGING STATUS

The indicator LED reflects the charging process:

- **Solid** = Charging Complete
- **Slow Pulse** = Charging
- **Fast Pulse** = Smart Mouthguard Is Misaligned In Charging Case

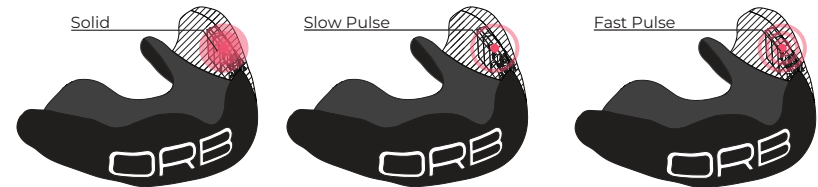


**NOTE:** If an ORB Sport mouthguard is being charged from a depleted battery, the red indicator LED may indicate misalignment, even if it is properly seated in the charging case. After 10 minutes, the LED should accurately reflect the charging status.

## 5

## BATTERY STATUS

When not actively charging or during a session, shake your ORB Sport mouthguard to show the battery charge:

- **1 Blink** = Low Battery
- **2 Blinks** = Medium Battery
- **3 Blinks** = High Battery

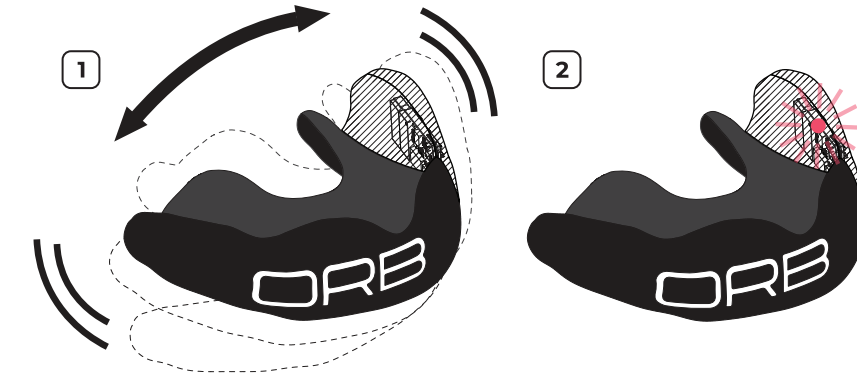


## 6

## MODES

The red indicator LED and optical heart rate sensor (OHR) indicate what state the ORB Sport mouthguard is in during use:

- **Off or Idle** - No LEDs are illuminated
- **Recording Data** - The OHR is illuminated constantly and the red indicator LED is pulsing
- **Download** - The OHR is off, and the red indicator LED blinks rapidly

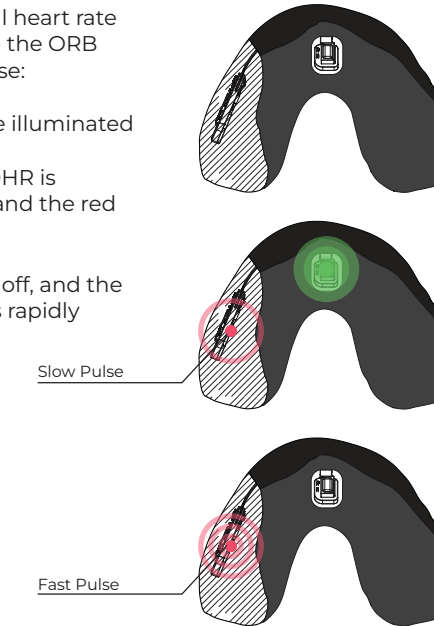

## 7

## PRODUCT SAFETY & CARE

**Please refer to the ORB Sport Instructions For Use for full safety and handling information.**

The ORB Sport mouthguard contains sensitive electronic components and can be damaged if dropped, chewed, burned, punctured, or subjected to forceful impact.



**!** Do not use a damaged mouthguard as it might cause injury.

Clean with cool water only (a soft toothbrush may be used) and leave to air-dry before charging.



**!** Ensure the mouthguard is dry before placing back into charging case.

## 8