

ORB SPORT™ SMART MOUTHGUARD CLINICAL REFERENCE GUIDE

Ready to order an ORB Sport™ Smart Mouthguard? Here's what you need to know to get started!

Identify a patient who qualifies.

- iPhone users with iOS 16 or later
- Active in sports that require or encourage mouthguards
- Athletes who desire valuable performance-and-wellness data

Send the patient's upper and lower impressions to Glidewell.

- Both digital and conventional impressions are accepted; visit glidewell.com/send-case for details
- For *digital impressions*, follow your scanner's recommended scan pathway to acquire an upper and lower full-arch scan, complete with a bi-lateral bite registration. Ensure that the upper scan captures the buccal gingiva as well as the full palatal anatomy.
- For *physical impressions*, utilizing an elastomeric impression material (no alginate submissions, please), capture the full upper and full lower arches, complete with a full arch or bi-lateral hard bite registration. Ensure that the upper impression captures the buccal gingiva as well as the full palatal anatomy.
- Allow for a four-day in-lab turnaround time

Fit the patient with their custom ORB Sport mouthguard.

- Ensure the patient's new appliance seats properly
- Minor chairside adjustments may be made to the mouthguard, taking care to avoid the electronics. Use caution while trimming or using a flame. Injudicious adjustment will risk damage to electronics and delamination of the mouthguard.

Give the patient the accompanying quick-start guide for setup.

- The patient will need to download the ORB Sport mobile app to pair the mouthguard and access data
- Be sure they benefit from the one-year **FREE** mobile app membership code available exclusively from you, their dental provider
- The patient will enter the offer code while signing up for the app
- For mobile app assistance, patients may contact support@orbsport.com or call **866-499-0058**

Cheer them on!

- The ORB Sport mouthguard is a valuable tool that will help your patient-athletes reach and maintain their maximum performance potential

Need more information?

If you or your clinical team has questions (e.g., seating, adjustments, warranty information), call Glidewell at **800-407-3326**.

If you've completed your in-office visit and your patient has questions (e.g., connecting to the mobile app, troubleshooting electronics, interpreting app metrics), have your patient call the athlete support line at **866-499-0058**.

NOTE: ORB Sport mouthguard is not designed or intended for use in the diagnosis, cure, mitigation, treatment, or prevention of any disease or condition.



Learn More
glidewell.com/orb-sport

ORB SPORT™
SMART MOUTHGUARD