

NEW!

BruxZir® NOW SRC



Quick Start Guide

Congratulations on choosing BruxZir® NOW SRC for a simplified approach to delivering same-day zirconia implant crowns.



1 Soak in alcohol then rinse with water.



2 Remove cover screw, healing cap, and provisional.



3 Rinse implant connection using chlorhexidine gluconate.



4 Place BruxZir NOW SRC and hand tighten.



5 Check contacts and occlusion, then take a bitewing x-ray.



6 Torque to 35 Ncm.



7 Seal using IvoClean®, Teflon tape, Scotchbond™, and a flowable composite.



8 Verify final occlusion using Shimstock strips or TrolFoil articulating foil.



9 If adjustments are needed, adjust and polish using the BruxZir™ Adjustment & Polishing Kit.

For more information
glidewell dental.com | 844-949-7184



for the sake of smiles

PK-5440900-121224